



PROTEOR  
**QUATTRO**<sup>™</sup>  
MICROPROCESSOR KNEE

# Prosthetist **Setup Guide**

Alignment and Setup | *For use with  
GaitLab App*



1

Download the GaitLab App  
from the Apple App Store or  
Google Play Store.



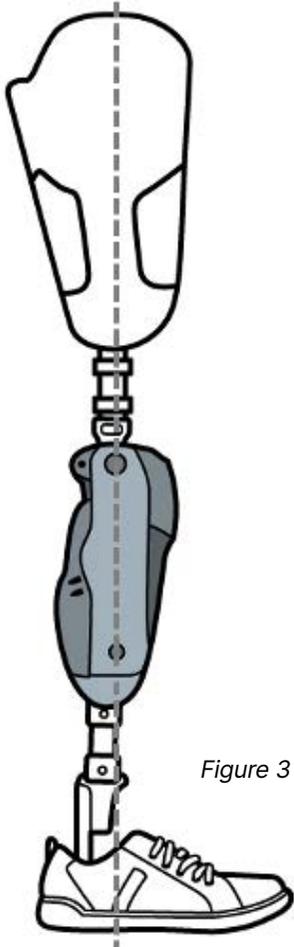


Figure 3

- 4 Follow the instructions in the App for bench and static alignment. Recommendation: A weight line that falls up to 10 mm anterior to the axis of rotation promotes stability. A weight line that falls up to 10 mm posterior to the axis of rotation may introduce stance flexion while walking.
- 5 Follow the instructions in the App to make adjustments to the knee. For each resistance change, sliding the setting to the left will decrease resistance and sliding the setting to right will increase resistance. Adjust until optimal setting achieved.
- 6 Follow next screens to demonstrate Sitting Mode, Dead Battery Mode, and Flexion Lock.
- 7 The tool bar across the bottom of the screen provides short cuts to the different functions of the knee.



Setup



Disconnect



Tools



Status



Help

**Setup:** Allows prosthetist to go through Full Setup Process, or individually Calibrate, Adjust resistance settings, or demonstrate to the patient the different modes available.

**Disconnect:** Allows prosthetist to disconnect from knee or unpair the knee and disconnect

**Tools:** Allows prosthetist to generate cadence report, update firmware or reset to factory defaults

**Status:** Shows knee connected, battery status, total steps, serial number, part number and firmware version

**Help:** Provides contact information for technical support

### Sitting Mode

With Sitting Mode turned on (default), when the knee is unweighted and the thigh reaches a 90 degree angle or greater, the knee flexes without resistance. With Sitting Mode off, the knee will continue to have hydraulic resistance to flexion independent of thigh flexion angle. To turn on/off select “Tools” tab in the GaitLab and Proteor app. Note: if Sitting mode is turned off, it will also disable Cycle Mode and Obstacle Assist.

### Obstacle Assist

Obstacle Assist can be turned off or on during full programming, and in the GaitLab App under “Setup” and “Adjust” after full programming is completed. With Obstacle Assist turned on (default), the knee will flex with no resistance when the thigh reaches 35 degrees of flexion or greater and is unweighted. This is helpful for stepping over tall objects or entering a high vehicle. When Obstacle Assist is turned off, the knee will flex slowly with hydraulic resistance when unweighted independent of thigh flexion angle. Note: if Sitting Mode is turned off, it will also disable Obstacle Assist.



### Stance Lock

The QUATTRO features a Stance Lock for stability on a flexed knee. When the user stops the thigh at a flexion angle and weights the knee, it will automatically lock further flexion. The lock will automatically disengage when the thigh moves in an extension moment or the knee is unweighted.

**Note:** *Stance Lock is available from 0-70° of flexion.*

### Transferring Settings and Resetting to Factory Default

1. Connect to the knee you'd like to transfer the settings *from*. Once connected, go to the “Tools” tab along the bottom of the App, then tap “Manage Settings”. Tap “Copy settings” to save the settings to your device. You can now disconnect from the programmed knee.
2. Connect to the knee you'd like to transfer the settings *to*. Go to the “Tools” tab along the bottom of the App, tap “Manage Settings”, and tap “Paste settings”. You will get a confirmation message “Are you sure you want to replace all settings?” Tap “OK” to transfer all settings, and “Cancel” to cancel the paste and keep the settings as is.
  - **Reset to factory defaults:** This will reset all settings in the knee to factory settings
  - **Recommendation:** Have the patient sitting while transferring the program or resetting the knee for safety.



1236 West Southern Avenue #101, Tempe, AZ 85282 855.450.7300

shop.proteorusa.com



#QUATTROKnee  
#HumanFirst

**#HUMANFIRST**